

# Amneal Pharmaceuticals

## Luer Cover Removal

The Amneal prefilled syringe features a **tilt** removal for the luer cover. **Do not twist** to remove as you may with other V-OVS closure systems.



Hold syringe upright on the ribbed part.



With the other hand, carefully tilt cap back and forth (DO NOT TWIST CAP) until it disconnects.



Pull cap straight upward to remove.

DO NOT TOUCH THE STERILE SYRINGE TIP.

For questions, contact Amneal at [DrugSafety@amneal.com](mailto:DrugSafety@amneal.com) or 877.835.5472, option 3